

Rx Only

For topical use only
73661-424-20

ADVANCED FORMULATION

Strata

scar therapy gel

triz[®]

For professional
scar therapy



ONCE
DAILY

Stratpharma
Switzerland

Federal Law restricts this device to sale by or on the order of a licensed healthcare practitioner.

The impact of scars



Scars are among the most common and frustrating results after a skin injury (i.e. surgery, accident, disease) and can have a physical as well as a psychological impact.¹

Abnormal scars can cause unpleasant symptoms and can be aesthetically disfiguring and distressing.²

Scars may cause **physical problems** like:²

- Itchiness, tenderness, pain and discomfort
- Remaining red/dark and raised without fading
- Movement limitation
- Functional disabilities
- Scar contractures (a wound that crosses a joint or a skin crease)

At times, the **psychological effects** can outweigh the physical ones for even the smallest of scars. For some people, a scar is a reminder of the traumatic event that caused it, which can lead to:²

- Disruption of daily activities
- Sleep disturbance, anxiety, depression
- Distress
- Loss of self-esteem, stigmatization
- Difficulties in social situations
- Diminished quality of life

To date, scars cannot be removed completely, but with effective treatment and good management, the appearance and side effects of a scar can be improved significantly. In addition, the likelihood of developing abnormal scars can be reduced effectively if treatment starts immediately after wound closure.

Why do we scar?

Scars are the natural outcome of the wound healing process.

The more and the deeper the skin is damaged, the longer and more complicated the rebuilding process – and the greater chance of developing a noticeable scar.

A **normal scar** usually develops during the first 48 hours after wound closure and fades in 3 - 12 months with an average time of 7 months.³

An active **abnormal scar** can grow bigger, remain red/dark, and may cause physical and psychological issues.

Abnormal scarring occurs more often in **higher-risk groups** which include people with darkly pigmented or very fair skin, those who have previously suffered from abnormal scarring, during hormonal changes (e.g. puberty, pregnancy) or when the wound or injury is more severe (e.g. burns, major trauma).

Common types of abnormal scars⁴



Linear hypertrophic scars are red, raised and sometimes itchy. They are confined to the border of the original surgery or trauma. They mature to an elevated with increased width. Full maturation can take up to two years.



Atrophic scars are flat and depressed below the surrounding skin. Generally small and often round with an indented or inverted center. Commonly arise after acne or chickenpox.



Keloid scars are raised, itchy scars that extend over normal tissue. They can continue to grow up to several years after injury and do not regress without treatment.



Widespread hypertrophic scars are common after a burn. They are widespread red, raised and sometimes itchy that remain within the borders of the original burn. Can develop contractures if they cross joints or skin creases.



Widespread stretched scars appear when a surgical scar gradually becomes stretched and widened. Typically flat, pale, soft and symptomless scars. No elevation, thickening or nodularity which distinguishes them from hypertrophic scars.

StrataTriz - professional scar therapy

StrataTriz is a transparent gel formulation for the management of scars, both old and new.

StrataTriz is intended to be used under the direction of healthcare practitioners in the management of scars.

StrataTriz dries to form a durable film that ensures constant contact with the scar.

StrataTriz is indicated for the use on all types of scars including those resulting from:

- Surgery
- Trauma
- Chronic wounds
- Burns
- Bites
- Acne and other diseases of the skin.

Clinical results

Linear hypertrophic scar after nevus extraction - chest⁵

Age of scars before treatment: 14 months

Left scar remained untreated, right scar treated for 2 months



Linear hypertrophic scar after trauma - head⁵

Age of scar before treatment: 2 weeks



SOFTENS AND FLATTENS

StrataTriz softens and flattens raised scars.



SYMPTOMATIC RELIEF

StrataTriz relieves itching and discomfort.



REDUCES REDNESS AND DISCOLORATION

StrataTriz reduces redness and discoloration associated with scars.



SCAR MANAGEMENT

StrataTriz helps to normalize the collagen synthesis cycles, producing a normal mature scar.



HYDRATION

StrataTriz is gas permeable and waterproof, which allows the skin to breath and remain hydrated.



PROTECTION

StrataTriz protects the scar from chemical and microbial invasion.



NON-REACTIVE

StrataTriz is inert, has no measurable pH, contains no alcohol, parabens or fragrances, making it suitable for children, people with sensitive skin, pregnant women and breastfeeding mothers.



COMPATIBLE

Once dry, StrataTriz can be covered by sunscreen, cosmetics, pressure garments or casts.



EASY TO USE

StrataTriz is convenient, easy to apply on all areas of the skin, including exposed areas as well as joints and hairy areas, and should be applied once daily.*

How to use StrataTriz



1. Ensure that the scar is clean and dry.



2. Apply a **very thin layer** of StrataTriz to the scar and allow to dry. When applied correctly, Strataderm should be dry in **3-4 minutes**.



3. If it takes longer to dry you have probably applied too much. Gently remove the excess with a tissue and allow the drying process to continue.



4. Once dry, StrataTriz can be covered by sunscreen, cosmetics, pressure garments or casts.

Additional directions

- StrataTriz should be applied once daily, or twice daily to exposed areas or as required to maintain contact with the affected surface.
- For best results StrataTriz should be maintained in continuous contact with the skin (24 hours a day/ 7 days a week).
- StrataTriz is recommended as minimum treatment of 60-90 days. Larger and older scars may take longer. Continued use is recommended until no further improvement is seen.
- StrataTriz does not need to be rubbed in or massaged, as it does not penetrate through the skin and will not enhance its effect.
- If not completely dry, StrataTriz may stain clothing. Normal washing will not remove the product from the clothes. If staining occurs, dry cleaning should be able to remove it without any damaging of the fabric.
- StrataTriz should not be applied over topical medications unless advised by the physician.

Dosage and administration

StrataTriz is an advanced formulation that requires substantially less product per application than typical creams or gels.



One 0.7 oz (20 g) tube is enough to treat an area of 5x5 inch (12x12 cm) for 30 days.

Additional scar care recommendations

Exposure – Avoid sun exposure and extreme temperatures.

Chemicals – Scars are very sensitive. Do not use products which contain strong chemicals (e.g. detergents, perfumes, herbal, botanical extracts, etc.)

Clothing – Avoid tight clothing to prevent irritation.

Activities – Avoid exercises or activities that cause tension on your scar.

Warnings and Contraindications

Warning: For external use only. StrataTriz should not be applied on third degree burns or to open wounds. StrataTriz should not be placed in contact with mucous membranes or the eyes. StrataTriz should not be applied over topical medications unless advised by your physician. StrataTriz may stain clothing if not completely dry. If staining occurs, dry cleaning should be able to remove it without damaging the fabric. For correct storage please reclose the tube tightly with the cap. If irritation occurs, discontinue use and consult your physician. Keep out of the reach of children. Do not use after the expiration (EXP) date printed on the tube. The expiration (EXP) date does not change once the tube has been opened. Do not use if the tube is damaged.

Contraindications Do not administer to patients with known hypersensitivity to the ingredients of this product.

PAY NO MORE THAN \$35*

ADVANCED FORMULATION
StrataTriz[®]
scar therapy gel

Rx Only
For topical use only
73661-424-20

Receive the lowest co-pay possible for your StrataTriz prescription.

If your prescription can be filled in the institutional pharmacy, please share the additional information with the pharmacist:

Processor: Drex
Bin#: 017290
RxPCN#: 55101202

Person Code: 01
Cardholder ID#: 1001001
Group#: E7000



Otherwise **your prescription will be filled by Truepill Pharmacy.** Your physician will send your prescription to Truepill Pharmacy along with your phone number. Expect a phone call or text from the pharmacy to establish contact with you, so they can collect the additional information needed to process your prescription.

Truepill

We are processing your prescription. What's next?

1. IMPORTANT

Truepill will call or text you to confirm your order. Answering the call or following the instruction in the text is important as they can't fill your prescription without this step!

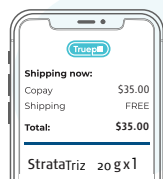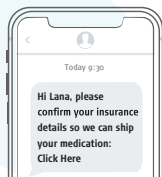


2. PAYMENT CONFIRMATION

Confirm the payment on the call or follow the instructions on the text. Truepill will work with your insurance and add your co-pay card to get you the lowest co-pay possible.

3. SHIPMENT CONFIRMATION

Truepill will call you to confirm the shipment or send you text with shipment information.

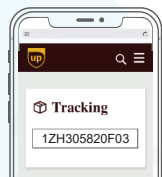

To call Truepill about your prescription: 1-855-910-8606

*most commercially insured patients

us.stratatriz.com

Ingredients: Polydimethylsiloxanes, siloxanes, alkylmethyl silicones. STERILE UNTIL OPENED.

References: 1. Atiyeh, B.S. (2007), *Aesthetic Plastic Surgery*, 31(5), pp. 468-492. 2. Bayat A et al. (2003), *British Medical J*, 326, pp. 88-92. 3. Bond J, et al. (2008), *Plastic and Reconstruction Surgery*, 121(2), pp. 487-496. 4. Bayat A et al. (2006), *British J of Hospital Medicine*, 67(8), pp. 634-639. 5. Data on file, 2010 (Dr. A. Irwin study). Stratpharma AG.



Class I Medical Device
FDA Listed

Distributed by: Distributed by: Marnel Pharmaceuticals
LLC Charleston, SC 29403, U.S.A. 1-833-788-0312

Stratpharma
Switzerland

ST-US-013-1-0820